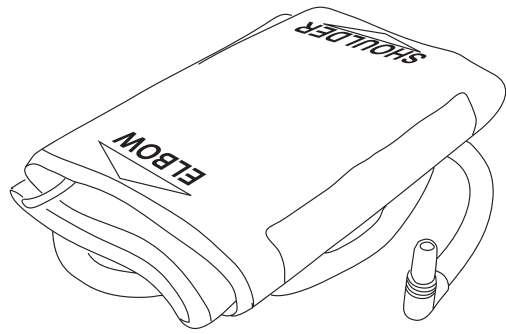
**ACTIVE & Duo ultima Series  
Reusable NIBP Cuff /  
ACTIVE & Duo ultima Serie  
Wiederverwendbare NIBP-Manschette /  
Brassard réutilisable NIBP gamme  
ACTIVE & Duo ultima /  
Serie ACTIVE & Duo ultima Bracciale  
riutilizzabile NIBP**

**Owner's Manual / Bedienungsanleitung /  
Notice D'utilisation / Manuale Dell'utente**

**ForaCare Suisse**  
Neugasse 55, 9000 St. Gallen, Switzerland  
www.foracare.ch

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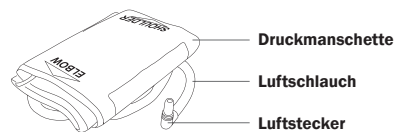
**EN** Thank you for purchasing FORA Reusable NIBP Cuff. Before you begin, please read this manual and the blood pressure monitor instruction manual thoroughly.

**SAFETY INFORMATION**

- Use this cuff **ONLY** for the intended use described in this manual.
- Do **NOT** use accessories which are not specified by the manufacturer.
- Do **NOT** use the cuff if it is not working properly or damaged.
- Do **NOT** use under any circumstances on newborns, infants, or persons who cannot communicate.
- Keep the cuff and its flexible cord away from hot surfaces.
- Do **NOT** apply the cuff to areas other than the place directed.
- Do **NOT** disassemble the cuff and its parts (ie. Airbag, air tube or air plug). Damages or substitution of a component different from that supplied might result in measurement error.

If you need further assistance, please contact your local customer service for help. If you experience any serious incident that occurred in relation to the use of this product, please report it to the manufacturer and the competent authority of medical devices in your country.

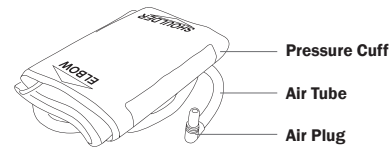
A serious incident means any incident that directly or indirectly led, might have led, or might lead to any of the following:

- the death of a patient, user, or other people,
- the temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- a serious public health threat.


**INTENDED USE**

FORA Reusable NIBP Cuff is intended to be used with FORA ACTIVE & Duo ultima series blood pressure monitoring system to measure the systolic and diastolic blood pressure and pulse rate. The system uses a non-invasive technique in which an inflatable cuff is wrapped on the upper arm. Do not use this system on babies, young children or persons who cannot express their consent.

**PRODUCT OVERVIEW**


**FITTING THE CUFF**

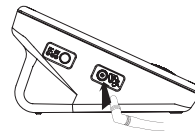1. Connect the air plug of the tubing to the air jack of the device.
2. Assemble the cuff. The smooth surface should be inside the cuff loop and the metal D-ring should not touch your skin.

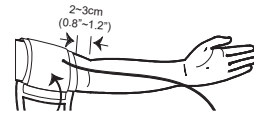**NOTE:**  
Before using the cuff, be sure the cuff size fits with your upper arm circumference.

3. Stretch your left (right) arm in front of you with your palm facing up. Slide and place the cuff onto your arm with the air tube and artery mark region (in red) toward the lower arm. Wrap and tighten the cuff above your elbow. The red line on the edge of the cuff should be approximately 0.8 to 1.2 inches (2 to 3 cm) above your elbow. Align the tube over the main arteries on the inside.

4. Leave a little free space between the arm and the cuff. You should be able to fit 2 fingers between them.

5. Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.

**PROPER MEASUREMENT POSITION**

1. Sit down for at least 10 minutes before measuring.
2. Place your elbow on a flat surface. Relax your hand with the palm facing up.
3. Make sure the cuff is about the same height as the location of your heart.

**ATTENTION!**

- Please remove your jacket or sweater before the measurement.
- Use a cloth slightly soaked with water to clean the cuff. Do **NOT** rinse or wash the cuff.
- Do **NOT** twist or disassemble the cuff or the air tube.
- Please replace the cuff with a new one when it is broken or when there are recurring inflation or deflation errors.
- Stop the use of the cuff if it causes irritated skin.

**SPECIFICATIONS**

**Model no.:** ACS012  
**Cuff Measurement Range:** 24 ~ 43 cm (9.4" ~ 16.9")  
**Dimension & Weight:** Cuff: L63.5 x W15 cm (25" x 5.9")  
Tube length: 80 cm (31.5")  
Weight: 132 g  
**Atmospheric Pressure Range:** 700 hPa to 1060 hPa  
**Operating Condition:** 5 °C ~ 40 °C, RH 15% ~ 93%  
**Storage Condition:** -25 °C ~ 70 °C, RH 10% ~ 95%

**WARRANTY TERMS AND CONDITIONS**

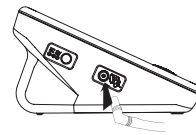
With respect to disposable products, ForaCare Suisse warrants to the original purchaser that, at time of delivery, each standard product manufactured by ForaCare Suisse shall be free from defects in material and workmanship and, when used for the purposes and indications described on the labeling, is fit for the purposes and indications described on the labeling. All warranties for a product shall expire as of the product expiration date, or if none, after one (1) year from the original date of purchase, as long as it has not been modified, altered, or misused.

ForaCare Suisse warranty hereunder shall not apply if: (i) a product is not used in accordance with its instructions or if it is used for a purpose not indicated on the labeling; (ii) any repairs, alterations or other work has been performed by the buyer or others on such item, other than work performed with ForaCare Suisse's authorisation and according to its approved procedures; or (iii) the alleged defect is a result of abuse, misuse, improper maintenance, accident or the negligence of any party other than ForaCare Suisse.

The warranty set forth herein is conditioned upon proper storage, installation, use and maintenance in accordance with applicable written recommendations from ForaCare Suisse. The warranty furnished hereunder does not extend to damaged items purchased hereunder resulting in whole or in part from the use of components, accessories, parts or supplies not furnished by ForaCare Suisse.

Use only with **FORA DUO ultima D40 / D40 Pro / ACTIVE plus P30 Plus / ACTIVE WIRELESS plus P30 Plus BT / FORA Vital** blood pressure monitoring systems.

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